

**1 – Freeze:** Press this button to freeze the image on your screen. Press again to go back to live scanning.

**2 – Zoom:** The plus and minus button allows you to zoom in and out on your screen in live scanning mode.


**3 – Wi-Fi:** This button will turn your Wi-Fi on or off. If the green light above the button is on, your Wi-Fi is on. Keep in mind, your machine will be factory set to have Wi-Fi turned off. Turn Wi-Fi on if you are connecting to a tablet. If you are not connecting to a tablet, leaving your Wi-Fi off will increase your battery life.

**4 – Exam:** Your ultrasound is preset with 4 different exams, with custom settings for different types of scanning. This button will toggle through the exams. Note; check your settings chart for a guide to the exams.

**6 – Power Button:** Press this button to turn the machine on/off. Note; you may need to hold the button for a couple seconds to make the machine turn on or off. When the machine is on, the light above the menu button (#5) turns green.



**5 - Menu:** Pressing this button will bring up the right menu, which allows you to adjust image settings like gain, dynamic, frequency, etc. Use the exam button to scroll through the menu, and the +/- buttons to adjust the settings. Press the menu button a second time to bring up the left menu which will allow you to adjust your grid, wireless channel, etc. Press the menu button a third time to exit the menu screen. See more videos online for settings help.

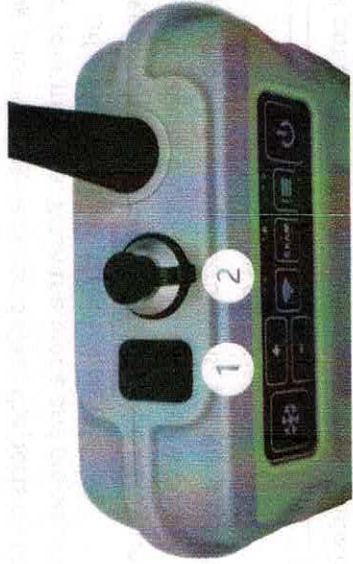

**1 – Charging Port:** The charging port is on the top of the ultrasound, all the way on the left. Lift the flap from the front of the machine to expose the charging port. Keep the flap closed and sealed when not charging the ultrasound to keep water, dust, and debris out of the charging port. Light above the exam button (#4) blinks when charging and is solid when charged. **Only use chargers provided by ReproScan to charge your ultrasound!**

**2 – VGA Port:** The round insert between the charging port and the probe cord is the VGA port. Lift the seal to expose the port to plug in your 2.0 monitor cable or goggles. Make sure to line up the red dot on the cable with the red dot on the VGA port.

Scan The QR Code for Apexx Start Up page that has the manual!



# APEX BY REPROSCAN

## FEATURES

The Apex ultrasound features 3 output signals:

### Direct Wired

Use the VGA port on the top of your ultrasound to plug in your 2.0 monitor or goggles.

### Wi-Fi

Your Apex is also equipped with Wi-Fi. Download the Apex app from the app store on your phone or tablet, connect your device to the Apex Wi-Fi network, and open the app to view your ultrasound image.

### Wireless

Apex and the 2.0 monitor can connect wirelessly. Make sure to change the input on your 2.0 monitor to WL-AV by pressing the "pawprint" button on your monitor.

### Interchangeable battery

The Apex is equipped with a battery door on the bottom of your ultrasound to make it easy to swap batteries and continue ultrasounding, even if you don't have access to power. Lift the metal ring up so that it is vertical, twist a quarter turn counterclockwise, and the battery door will open. Swap your battery out, replace the cover, turn the metal ring back to its original position. If you would like to purchase an extra battery, contact your ReproScan representative.

### **Tips and tricks to get the most out of your ultrasound:**

#### Use Your Exams

Your machine comes with 4 preset exams, with settings that are optimized for different types of scanning. Check your settings chart for a guide to the different exams.

#### Keep Your Port Covers Closed

There are covers for you VGA and charging ports. When you don't have anything plugged into these ports, keep the covers closed. This will help keep water, dust, and debris out of the ports.

#### Clean Your Machine

When you are done scanning, turn your machine off, unplug your viewing device and close the ports on the top of your machine. Wash the probe and probe cord off with water. Use paper towels or a soft sponge/brush to get manure off. Wipe the ultrasound off with a damp paper towel or cleaning wipe. For more detailed cleaning instructions, see the product manual or follow us on social media.

Check out our YouTube Page for some helpful videos!



Contact your ReproScan representative at  
877-890-2411 for any further questions!

# APEX BY REPROSCAN